EN Instructions for Use

Daily disposable – replace every 24 hours
The geko™ device is a neuromuscular electro-stimulation device. CE Marked as a medical device, and is intended use is to increase blood circulation

- for the prevention of venous thrombosis
 - for the prevention and treatment of oedema
 - for the treatment of venous insufficiency and ischemia
- geko™** is intended for use on a single patient as part of a single course may be removed immediately and reapplied if necessary, for example to prevent the device getting wet during bathing or showering.

Positioning
The device works by stimulating the common peroneal nerve which activates the calf and foot muscle pumps to increase blood circulation.

Positioning the blue marker band over the head of the fibula

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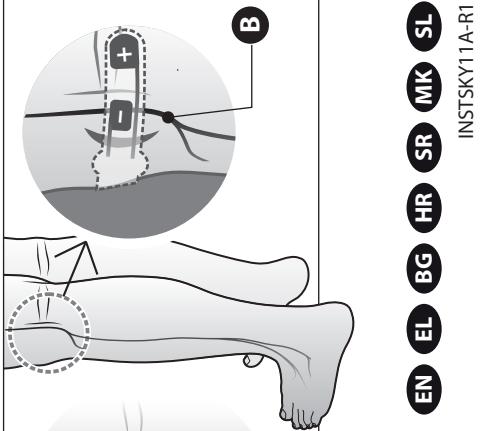
2. Assess the need for exfoliation: this may only be required if the outer skin layer (stratum corneum) is particularly thick. If required, using the grey abrasive pad, gently exfoliate the skin. The skin should be sufficiently moist for the procedure, so as a wet should be sufficient for this procedure. DO NOT over exfoliate, especially if the skin is fragile or may be damaged through abrasion) and then work thoroughly with the electrode preparation wipe. For best results, remove any moisturiser, and any excess hair using trimmers or clippers (avoid shaving as this may damage the skin).

3. Peel off the protective film from the long end first and secure the device to a straight leg so that the location markings are positioned directly over the centre of fibula head.

Next, remove the protective film from the short end and secure this towards the front of the leg. (Correctly fitted, the device should not be felt like a band around the fibula bone, which will cause discomfort.) If you are unsure how to locate the fibula head or the common peroneal nerve, ask your healthcare provider (e.g. nurse practitioner, physiotherapist, G.P., physician, or surgeon).

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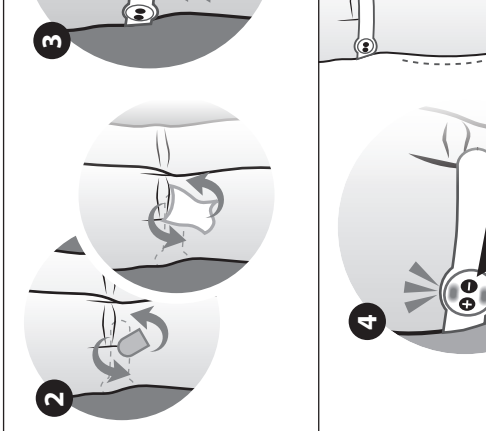
4. Use a short piece of the t-shirt (0.5 second) to turn on the device and further about 20 seconds to set the stimulation level: this is reached when there is a visible movement of the muscles in the lower leg, indicated in Fig. A and B (stimulated muscles) overlap, moving the foot upwards.

To accommodate any variation in stimulus due to leg position changes, simply turn the setting up or down. There are 7 levels (indicated by the number of flashes in sequence from the LED while the device is operating); in most cases levels 5 or 6 are ideal. To reduce the effects, turn the device off for 10 seconds at the time – “fasten”.

To turn off the device or “fasten” the treatment, hold the ‘-

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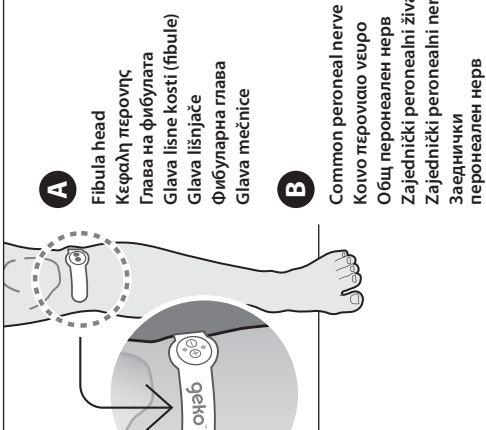
muscles, stimulation may be achieved using the alternative location behind the knee; after preparing the skin (step 2), apply the device either slightly above or below the crease at the tendon. See [www.gekodevices.com](#) for further details

The setting required to achieve stimulation may be affected in hot or anaerobic environments. If the device is not stimulated more easily, readjust the setting in the stimulation tent. See [www.gekodevices.com](#) for further details

The device should be fitted to conform to the leg. If there is ensuring that the marker line is still correctly aligned with the fibula head.

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EN

Warnings

Implanted electronic devices (for example a cardiac pacemaker): - specialist medical opinion should be obtained before the patient is subjected to stimulation. Recently diagnosed or suspected DVT; specialist medical opinion should be obtained before the patient is subjected to stimulation.

Read the instructions carefully before use. Only apply the device in accordance with instructions. Improper use of this device can be dangerous.

The geko™ device may be operated either by the patient or a third party. Patients should be trained in the positioning and operation of the device.

Keep out of the reach of children and pets.

No modification of this equipment is allowed.

Do not use whilst driving, operating machinery, or during any activity in which involuntary muscle contractions may put the user or others at undue risk of injury.

Stimulation should not be applied across or through the head, directly on the eyes, covering the mouth, or from the front of the neck, (especially the carotid sinus), or from electrodes placed on the chest and the upper back or crossing over the heart.

Application of electrodes near the thorax may increase the risk of cardiac fibrillation.

Do not apply over sore, infected or inflamed areas, broken skin or skin eruptions, e.g. phlebitis, thrombophlebitis, varicose veins etc.

Do not apply over or, in proximity to, cancerous lesions.

Do not use in proximity (i.e. within 1m) of short wave/ microwave equipment, as this may affect the device.

Portable and mobile RF communications equipment can affect medical electrical equipment.

Do not use if packaging is open or appears damaged.

Do not use a device that appears to be damaged.

Do not use the device in close proximity to heat sources, such as fires or radiant heaters, as excess heat may affect the performance of the device.

In some cases skin inflammation or irritation can develop in the contact area: either remove the device or re-attach in the alternative fitting location. If the condition persists or recurs obtain specialist medical advice before resuming use.

Medical advice should be sought when the patient is pregnant, has diagnosed heart conditions or epilepsy, or following surgery where muscle contractions may disrupt the healing process.

The device should be removed before the patient

undergoes MRI as it contains ferromagnetic components.

The device should be switched off during ECG monitoring using leg electrodes as it may interfere with ECG leg electrode signals.

The device is not intended for use in an oxygen rich environment.

Precautions

Check the device is functioning correctly every 2 hours. Use the device only as instructed. Visit www.gekodevices.com for further advice and usage tips. Keep out of the reach of children.

Do not switch on unless correctly attached to the patient.

The device must be kept clean and dry. Do not bath or shower whilst wearing the device.

Do not apply a device that has been worn by another individual as this will carry a risk of infection.

Specialist medical opinion should be obtained before devices are used for more than 28 consecutive days.

Remove the device carefully, lifting the long-end first and applying a constant gentle force.

Excessive force may damage the device.

EMC Declaration

The geko™ device is certified to EN 60601-1-2:2007 regarding Electromagnetic Compatibility. Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided at: <http://www.gekodevices.com/>

Disposal

Batteries must be disposed of in accordance with any local legislation. The battery can be removed by breaking open the housing and prising out.

See www.gekodevices.com for guidance.

Do not incinerate the device.

Symbols

⊕	On/Increase stimulation button	⊖	Off/decrease stimulation button
----------------	--------------------------------	----------------	---------------------------------

Firstkind Limited*

Hawk House
Peregrine Business Park
High Wycombe
HP13 7DL
United Kingdom
+44 (0)1494 572040

www.gekodevices.com

*Firstkind Limited is a wholly owned subsidiary of Sky Medical Technology Limited.

If assistance is required or to report difficulties, unexpected operation or events, contact the manufacturer.

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